

Standard mesh:

Mix mild soap and warm water together and using a lint-free cloth, damp it a little bit into the mixture. Dab gently where the stain is located. Use a different, dry, lint-free cloth to dry the area. Repeat if needed.

Copper-infused mesh:

To prevent overall soil, frequently vacuum and lightly brush the best to remove dust and grime buildup. Spot clean with mild soap and water. Infrequent use of a 1:100 bleach to water solution can be used for disinfection of the mesh. The accumulate wet contact time should be limited to a maximum of two hours over the life of the chair. Regular bleach cleaning of this mesh is not recommended.

The following cleaners have been successfully used with copper-infused mesh. Please note, Sinfonia does not endorse any of these cleaners. For details, please contact these cleaning product manufacturers directly.

- | | |
|---|---|
| 1. CLOROX® Broad Spectrum Quaternary Disinfectant Cleaner | 9. SEALED AIR VIREX II 256 |
| 2. CLOROX® Healthcare bleach germicidal cleaner | 10. SEALED AIR AVERT™ Disinfectant Cleaner |
| 3. CLOROX® Dispatch | 11. SEALED AIR Whistle® All Purpose Disinfectant Tb Cleaner |
| 4. CLOROX® Clean-Up® Disinfectant Cleaner with Bleach | 12. SEALED AIR EXPOSE® II 256 |
| 5. ECOLAB ASEPTI-WIPE-II | 13. STERIS Coverage® Spray TB |
| 6. GOJO PURELL® Healthcare Surface Disinfectant | 14. STERIS Coverage® Plus NPD |
| 7. METREX Cavicide™ | 15. STERIS Coverage Spray Coverage® Plus Germicidal wipes |
| 8. METREX Envirocide™ | |

Standard upholstery:

Vacuum up any excess dirt/spillage with close attention to the seams, area where the seat meets the back and area where the arms meet the seat/back. After, take a lint-free cloth and dip it into any water-based cleaning agent. Scrub gently over the stained area. Repeat if needed. For best results, follow the mill guidelines for your exact textile.

To best maintain your casters:

To get the most out of your casters, it is always good to keep them as clean as possible. Once a month, gently flip your chair over and carefully use tweezers to remove all dirt/hair/debris out of the creases of the caster. Use a lint-free cloth with your preferred cleaning agent and clean the caster thoroughly. Thoroughly dry the caster with another lint-free cloth after.